
Instructions

- 1 Secure the inner cable to the motor using the set screw on the cable. You may need to pull the inner cable out slightly to expose the set screw.

Requires 3/32 Allen Wrench



- 2 Once the inner cable is attached, screw the black connector piece onto the motor. Use a crescent wrench to tighten the connector. This piece has reverse threads. It tightens to the left, and loosens to the right.



www.EquineDentalTools.com



Square Drive Cables



For More Tips and Information Visit
www.EquineDentalTools.com

JERRYs

Equine Dental Tools

Address:
9225 Rocky Canyon Road
Atascadero, CA 93422

Telephone:
805-460-7276

Email:
jerry@jerryseqdentistry.com



Suggested Maintenance

1 Inner cables can break if a horse bites down while the motor runs. We recommend keeping a couple extra on hand.

Replacement inner cables are not pre-greased. Apply a thin layer of grease before installing a new inner cable.

2

Approximately 1 inch of the inner cable should be exposed at the end of the outer sheath. If the cable is not centered within the outer sheath, it may not turn properly.

If your tool is need of
repair or maintenance,
send it in to our top-notch
repair specialist!

Warranty Information

Our tools are thoroughly inspected and tested in our machine shop. All new tools sold by Jerry's Equine Dental Tools are warranted to be free of defects from workmanship and materials for the period of 6 MONTHS from the date of the original purchase.

This warranty does not apply where:

- Repairs have been made or attempted by others
- Repairs are required because of normal wear and tear
- The tool has been abused, misused or improperly maintained
- Alterations have been made to the tool

IN NO EVENT SHALL JERRY'S BE
LIABLE FOR ANY INDIRECT, INCI-
DENTAL, PUNITIVE OR CONSE-
QUENTIAL DAMAGES ARISING
FROM THE SALE OR USE OF THE
PRODUCTS SOLD HEREUNDER