Instructions

Remove inner cable from outer sheath. Then insert the inner cable into the grey connector as shown.





Secure the inner cable to the motor using the set screw on the cable.

Screw the grey connector piece onto the motor.





Slide the outer sheath onto the cable until it reaches the motor.

Tighten the set screw on the grey connector to secure the cable.



www.EquineDentalTools.com




For More Tips and Information Visit www.EquineDentalTools.com

JERRYS Equine Dental Tools

Address: 9225 Rocky Canyon Road Atascadero, CA 93422

Telephone: 805-460-7276

Email: jerry@jerryseqdentistry.com

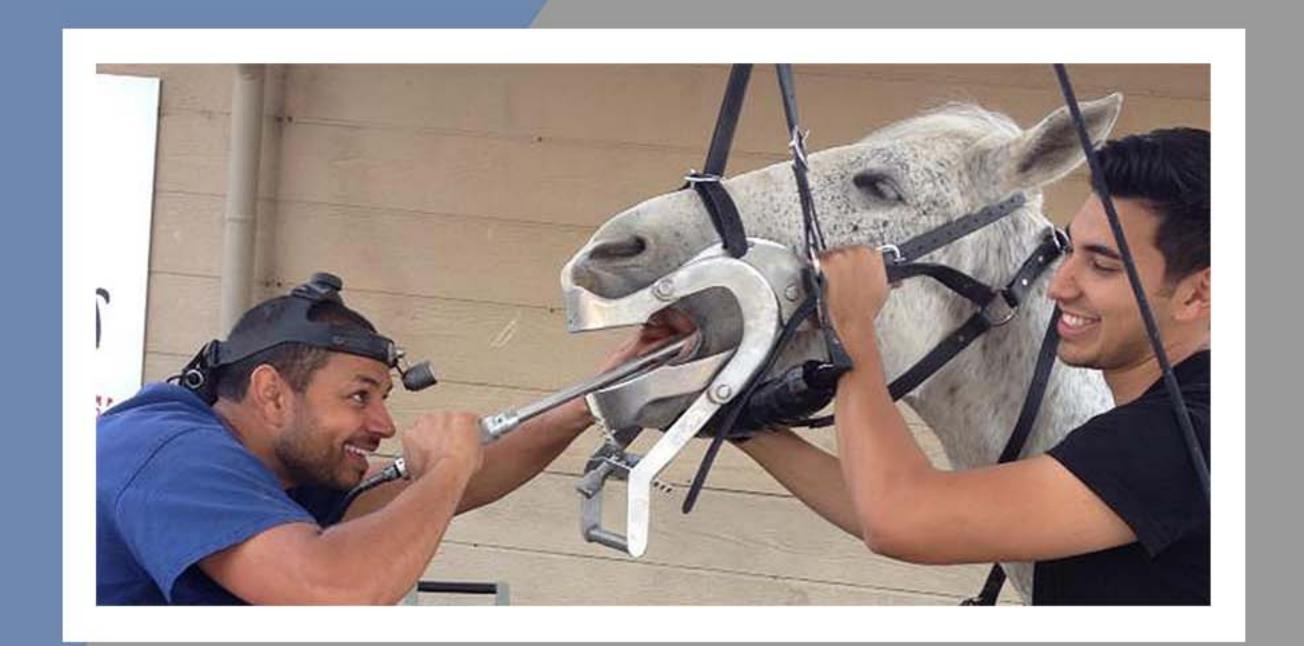

Suggested Maitenance

Inner cables can break if a horse bites down while the motor runs. We recommend keeping a couple extra on hand.

Replacement inner cables are not pre-greased.

Apply a thin layer of grease before installing a new inner cable.

Approximately 1 inch of the inner cable should be exposed at the end of the outer sheath. If the cable is not centered within the outer sheath, it may not turn properly.

If your tool is need of repair or maintenance, send it in to our top-notch repair specialist!

Warranty Information

Our tools are thoroughly inspected and tested in our machine shop. All new tools sold by Jerry's Equine Dental Tools are warranted to be free of defects from workmanship and materials for the period of 6 MONTHS from the date of the original purchase.

This warranty does not apply where:

- Repairs have been made or attempted by others
- Repairs are required because of normal wear and tear
- The tool has been abused, misused or improperly maintained
- Alterations have been made to the tool

IN NO EVENT SHALL JERRY'S BE LIABLE FOR ANY INDIRECT, INCIDENTAL, PUNITIVE OR CONSEQUENTIAL DAMAGES ARISING FROM THE SALE OR USE OF THE PRODUCTS SOLD HEREUNDER