
Instructions

1 To open the EZ spec, simply pull on the front strap. Pressing the strap clip will allow the speculum to close.

2 The EZ Spec can be used with the handle facing up or down. Placing the handle up will give you extra leverage when opening the speculum.

For Instructional Videos and More Visit
www.EquineDentalTools.com

www.EquineDentalTools.com



EZ SPEC

For More Tips and Information Visit
www.EquineDentalTools.com

JERRYs

Equine Dental Tools

Address:
9225 Rocky Canyon Road
Atascadero, CA 93422

Telephone:
805-460-7276

Email:
jerry@jerryseqdentistry.com



Suggested Maintenance

1 Be sure to keep the front strap free of dirt and debris. The EZ Spec can be cleaned with soapy water and a soft brush. We do not recommend leaving the speculum submerged in water.

With regular use, the bolts on the sides of the EZ Spec may loosen slightly. A wrench may be needed to tighten them occasionally.

2

If your tool is need of repair or maintenance, send it in to our top-notch repair specialist!

Warrnaty Information

Our tools are thoroughly inspected and tested in our machine shop. All new tools sold by Jerry's Equine Dental Tools are warranted to be free of defects from workmanship and materials for the period of 6 MONTHS from the date of the original purchase.

This warranty does not apply where:

- Repairs have been made or attempted by others
- Repairs are required because of normal wear and tear
- The tool has been abused, misused or improperly maintained
- Alterations have been made to the tool

IN NO EVENT SHALL JERRY'S BE LIABLE FOR ANY INDIRECT, INCIDENTAL, PUNITIVE OR CONSEQUENTIAL DAMAGES ARISING FROM THE SALE OR USE OF THE PRODUCTS SOLD HEREUNDER